INSTRUCTIONS FOR USE

CONTENTS

- **English** Version ----- 01 - 06
- Version **Française** ----- 07 - 12

INSTRUCTIONS FOR USE

English Version

Angel Aligner is indicated for the alignment of teeth during orthodontic treatment of malocclusion.

1. Product Overview

Angel Aligner consists of removable orthodontic aligners, and where needed accessories to help improve the treatment effects. Angel Aligner products are clear plastic appliances that achieve optimal results through a customized treatment plan including a series of aligners that move teeth in stages. Each stage corresponds to one or multiple Angel Aligner and the total number will vary depending on a clinician's treatment plan and case complexity.

Product: Clear Aligner

Product model: Pro, Select, Kid

2. Product Composition

The clear orthodontic aligners are all transparent appliances. The products are composed of aligner and attachment template (optional). They are commonly made by thermal pressure molding process using sheet material with a thickness of 0.1-3.0 mm. They can be made of transparent thermoplastic polyurethane material or transparent thermoplastic copolymer polyester-polyurethane multilayer composite material. The template is made of polyurethane, polyethylene or polyester-polyurethane multilayer composite materials. The inside surface of the product is designed to perfectly fit the shape of teeth while varies types of accessories can be applied on the outside surface following treatment plan.

3. Activation Mechanism

The treatment of Angel Aligner is achieved by planning the teeth movement digitally and moving the teeth through a series of planned target positions.

In clinical applications, a treatment plan commonly includes several stages as various teeth movements are needed at different stages of treatment. The treatment is divided into several stages and each stage corresponds to one or multiple Angel Aligner. To gently move the teeth to the target position of each stage, the Angel Aligner are designed to fit the shape of teeth crown. Each treatment plan is custom-made to account for a patients' oral condition, seriousness of malocclusion, patient's age, lifestyle, etc.

4. Instructions for Use

Your doctor's customized treatment plan is based on your doctor's prescription. Please read these directions carefully before use. Wear your aligners per your doctor's instructions, usually 22 hours a day or more.

a. Wearing procedure

1) Clean your aligners:

A soft-bristled toothbrush is recommended to clean your aligner. Dip the toothbrush in water to wipe the inner and outer surfaces of the aligner. Then rinse with water.

2) Clean your mouth:

Brush your teeth and rinse your mouth with water.

3) Clean your hands:

Wash your hands before handling your aligners.

4) Insert your aligners:

Make sure you have the correct aligner. The "U" denotes upper teeth or "L" for lower teeth. This is followed by a stage number. Example: "U 1" means the first stage of upper, and "U 1-A" means the first aligner of the first stage of upper teeth. Fit your aligner on anterior teeth first, then posterior teeth.

Procedure:

- First, the anterior area of aligner is pressed onto the anterior teeth area of the aligner.
- Then the posterior area of aligner is pressed onto the posterior teeth area.
- After the appliances are fit in, various types of occlusion can be conducted to perfectly fit the appliances with teeth.

If part of appliance and appliance are not perfectly fit, the patient can use this part of teeth to bite medical cotton roll, and help the appliance in good fit.

b. Removing procedure

Principle: Remove the appliance from posterior teeth to anterior teeth, from lingual to buccal region, and alternately on both sides. Procedure:

- Maxillary appliance: The maxillary appliance can be removed from one side to the other. First, clamp the lingual side of one side's posterior appliance with fingers. Then pull the lingual edge of posterior appliance downward and buccally simultaneously. Both lingual and buccal side of appliance are pulled alternately until the appliance is detached from the teeth.
- Mandible appliance: The mandible appliance can be removed either from one side to the other or both sides simultaneously. First, clamp the buccal side of one side's posterior appliance with fingers. Then pull the buccal edge of posterior appliance upward and buccally simultaneously. Both lingual and buccal side of appliance are pulled alternately until the appliance is detached from the teeth.

c. Method of preservation

The aligners should be removed during eating, brushing teeth and can be removed for short social occasions. The appliance should be kept in a cleaned case after cleaning. It needs to be cleaned again before use.

d. Wearing time

- 1) The appliance should be worn for 22 hours or more per day for 1-2 weeks, or as instructed by your doctor, to achieve effective tooth movement.
- 2) Each stage appliances should be worn for 1-2 weeks, or as prescribed by your doctor. Before moving to a new aligner, your present aligners should fully fit over the teeth, be easily removed and no orthodontic force should be felt. It is recommended to change into new aligners at night prior to bedtime.
- 3) If the patient is unable to wear the appliances as prescriptive information, the patient should contact their doctor as soon as possible. To adjust the prescriptive wearing time, the doctor's consent must be obtained to continue the treatment. The treatment plan shall be followed, or the treatment shall be deemed as self-termination.

e. Request for revisit

- 1) The patient shall follow a regular return visit schedule according to the doctor's request to investigate the function and progress of the treatment.
- 2) The current appliances should be worn at the time of return visit. The patient should retain the last 2 aligner stages and bring to return visits.

INSTRUCTIONS FOR USE

5. Safety Considerations

- a. The appliances shall be used under the guidance of the attending doctor, and the treatment plan for its use shall be formulated by the attending doctor.
- b. The appliances are made for a particular patient, at a particular stage and in a correct sequence. The appliances must be used in order determined by the orthodontics sequence and can only be used for that particular patient.
- c. In the use of the appliances for orthodontic treatment, the patient may experience a certain level of discomfort. This is considered normal. If there are questions or prolonged discomfort, the patient should contact the treating doctor immediately.
- d. If there is discomfort when wearing the appliances, patient should not adjust the appliances, but contact the doctor as soon as possible.
- e. After receiving the appliances, the patient shall wear it immediately to prevent unintended tooth movement.
- f. If the patient has one or more missing teeth, he or she must wear retaining appliances immediately.
- g. If the patient is unable to wear the Angel Aligner for the recommended time, does not comply with the doctor's instructions for wear, cannot complete return visits on time, there is the presence of abnormal shape of teeth, or there is new tooth growth during treatment, the orthodontic treatment and timing may be affected.
- h. Some medical conditions and the use of drugs may affect the treatment.
- i. Malocclusion, severe overbite, and mixed dentition may require treatment other than Angel Aligner.
- j. The combination of other fixed/active orthodontic appliances may be required for complex treatment options that are not fully competent with the Angel Aligner.
- k. Severe dentition congestion or occlusal imbalance may require the intervention of oral surgery. If this type of surgery is needed, the corresponding risks should be considered before treatment.
- l. Unconventional shapes, sprouting or missing teeth may affect the wearing of orthodontic appliances or may affect the treatment.
- m. The tooth with fillings or restorations may be displaced, and the fillings or restorations may need to be replaced.
- n. The appliances do not move the dental implant effectively.
- o. In cases of traction accessories, the patients are required to cooperate with the traction according to the doctor's instructions.
- p. The functional accessories, may fall off because of numerous reasons, including improper intraoral habits, malocclusion, improper usage, etc. When it happens, please contact your doctor as soon as possible. On rare occasions, the falling part may be swallowed or breathed in. In case it happens, please seek treatment as soon as possible.
- q. Please avoid affecting the use of appliance because of improper intraoral habits, malocclusion, improper usage, etc. When the appliance is fractured locally or damaged, please contact your doctor as soon as possible. When the surface of appliance is peeled off or abased, please also contact your doctor as soon as possible. For other cases or questions, please refer to your doctor.
- r. If the appliance cannot be engaged, please contact your doctor as soon as possible.
- s. In case of damage to the oral mucosa caused by wearing the appliances, the patient should contact the doctor as soon as possible and no self-adjustment shall be allowed.
- t. In the case of gum injury, oral ulcers and abnormal loosening of teeth caused by the non-appliances itself, patient should contact the doctor as soon as possible. If necessary, stop using the orthodontic appliances and wait for the tissue to recover before continuing to correct.
- u. When the appliances are damaged, deformed or lost, patient should contact the doctor as soon as possible and take prompt remedial measures.
- v. Angel Aligner with mandibular advancement features is neither tested nor approved for treatment of respiratory disorders, sleep apnea, and/or temporomandibular disorders (TMD).
- w. In rare instances, problems in the temporomandibular or jaw joint (temporomandibular disorder or dysfunction (TMD)) may result in joint pain, headaches, or ear problems. For patients in treatment with mandibular advancement features such as Occlusal Block (which are used to shift the lower jaw forward), problems in the jaw joint may be exacerbated.

6. Precautions

- Patient should not wear appliance to drink hot tea, drinks, etc.
- Patient should wear your aligners per your doctor's instructions, usually 22 hours a day or more.
- After eating or drinking sugary drinks, patient should take oral cleaning measures (such as gargling with water, brushing their teeth, etc.) before wearing the appliances.
- The aligner should not be soaked in hot water or disinfectant water to avoid aligner deformation.
- Patient should contact the doctor as soon as possible for the excessive irritation to the oral mucosa caused by wearing the appliances, and no self-adjustment by patient should be allowed.
- In the case of gum injury, oral ulcers and abnormal loosening of teeth caused by the appliance itself, patient should contact the doctor as soon as possible. If necessary, stop using the orthodontic appliances and wait for the tissue to recover before continuing to correct.
- Tobacco use, smoking, and/or vaping while wearing aligners may lead to discoloration or damage of the aligner.
- When the appliance(s) are damaged, deformed or lost, patient should contact the doctor as soon as possible. Replacements should be ordered as soon as possible.

7. Indications for Use

Angel Aligner are indicated for the alignment of teeth during orthodontic treatment of malocclusion.

8. Contraindications

Angel Aligners is contraindicated for patients with:

Serious progressive periodontal disease,

Known TPU/PETG/PE allergies,

Warning: Patients using Angel Aligners should be able to understand and follow any treatment instructions as provided by their doctor.

9. Storage Condition

The product should be stored in a non-corrosive gas, cool and clean environment.

10. Maintenance

After each use, a soft tooth brush to clean the inside and outside surfaces of the appliances, and then put the appliances into a clean case for storage. When the appliances are damaged, deformed or lost, patient should contact the doctor as soon as possible and take prompt remedial measures.

11. Product Wear Cycle

Angel Aligner are typically designed for 1-2 weeks wear, unless instructed otherwise by the doctor.

12. Label Description

The "U" in the "Stage" denotes the maxilla (upper); "L" means the mandible (lower). Example: "U 1" means the first stage of maxillary.

13. Waste & Disposal

The appliance is eco-friendly and does not contain environmentally harmful substances. When handling used appliances, please refer to your doctor and follow your local waste & disposal policy.

INSTRUCTIONS FOR USE

14. Date of Production

See product packaging label for details.

15. Term of Use

Under proper storage condition, the appliances have a shelf life of 3 years after produced.

16. Symbol Meaning

Symbol	Introductions	Symbol	Introductions
	Batch Code		Use-by date
	Caution		Consult instructions for use or consult electronic instructions for use
	Medical Device		Manufacture Date
	Manufacturer Name and Address	Rx only	Prescription Use Only
	Model Number		Non sterile
	Patient Number		Unique device identifier

Manufacturer: Wuxi EA Bio-Tech Co., Ltd.

Address: No.36 Guanshan Road, Xinwu District, Wuxi, Jiangsu Province, China 214131

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