INSTRUCTIONS FOR USE

CONTENTS

- **English** Version ----- 01 - 06
- Versión **Española** ----- 07 - 12

INSTRUCTIONS FOR USE

English Version

The invisible orthodontics aligner described in this instruction manual are for the alignment of teeth during orthodontic treatment of malocclusion. This instruction manual includes the product description, contraindications, precautions, warnings, instruction for use, recommended software, basic directions for dispensing, fit and follow-up and safety considerations for invisible aligners. It is specifically written for dentists who are qualified to perform clinical procedure and have completed all aspects of general dental training.

1. Product overview

Instead of metal brackets and wires, Clear aligners are clear plastic appliances that achieve optimal results through a customized treatment plan including a series of aligners that move teeth in stages. Each stage corresponds to one or multiple clear aligners and the total number will vary depending on a clinician's treatment plan and case complexity. The product family and their uses are shown in Table 1.

The appliances are a series of custom-made plastic removable aligners. Due to the variation of personal oral conditions and treatment plans, there is no standard dimension.

Table 1. The product family and their uses of invisible orthodontics appliance.

Product	Product specification
Orthodontic aligner	The orthodontic aligners are designed to move the teeth gently through a series of clear aligners. The aligners are made through a thermal pressure molding process that uses a plastic sheet material with a thickness between 0.1 and 3.0 mm.
Accessory	Various types of accessories can be added to help improve the treatment effects. These accessories are optional based on the clinical requirement; It can be specified as: 'angelButton' which is used for employ the traction and 'A6' which is used for mandibular advancement.

2. Product composition

The invisible orthodontic aligners are all transparent appliances. They are commonly made by thermal pressure molding process using sheet material with a thickness of 0.1-3.0 mm. They can be made of transparent thermoplastic polyurethane material or transparent thermoplastic copolymer polyester-polyurethane multilayer composite material. For each stage, the aligners made from various material can be combined used.

The inside surface of product is designed to perfectly fit the shape of teeth while varies types of accessories can be applied on the outside surface following treatment plan. Traction accessory is a traditional procedure and a major type of accessories to assist and improve clinical effects. The traction accessory designed by Wuxi EA Medical Instruments Technologies Limited is made from thermoplastic polyurethane materials. It is bonded to the outer surface of the product by adhesive to engage varies types of tractions. The adhesive used includes acrylic polyurethane, etc.

3. Activation mechanism

The treatment of invisible orthodontics appliances is achieved by planning the teeth movement digitally and moving the teeth to a series of planned target positions. In clinical applications, a treatment plan commonly includes several stages as various teeth movement are needed at different stages of treatment. The treatments are further divided into several stages and each stage corresponds to one or multiple invisible aligners. The aligners can be made from various material and combined used. To gently move the teeth to the target position of each stage, the orthodontics aligners are designed to fit the shape of teeth crown at target position. When the aligners are worn, the teeth are not in the target position. The difference between teeth position and target position would cause elastic deformation of aligners, which yield dynamics to move the teeth and achieve orthodontic plan. By designing a series of target position and fabricating a series of corresponding aligner, a series of target teeth movement can be gently achieved. It is worth noting that the treatment plan is custom-made due to variety of patients' oral condition, seriousness of malocclusion, patient's age, lifestyle, etc. With custom-made treatment plan, series of custom-made orthodontics appliance are designed and manufactured.

A6 Angel Align aligners are a treatment option that adds twin mandibular repositioning features to the posterior teeth for mandibular advancement. The A6 repositioning features are added in the form of matching blocks on each upper lower aligner tray. These blocks function as the mechanism of action to position a patient's mandible, maintaining the mandible in a more optimal bite position by applying occlusal forces in occlusal inclined planes. The blocks are commonly placed symmetrically to avoid forces in non-occlusal planes and have a textured non-slip pattern to maintain effective occlusal forces. The occlusal block in the maxillary aligner covers two teeth while the one in the mandible aligner covers only one tooth.

You, as the treating physician, can choose the location of the features and the occlusal block can be applied on the first and second premolar or second premolar and first molar. The A6 blocks can be designed in two configurations: "Type I", designed for patients with suitable space in the posterior teeth region and patients with more level Spee curves (occlusal curvature), and is optimized for compressive strength. The second configuration, "Type II," will have a customized posterior area to account for patients with deeper Spee curves in order to achieve optimal forces to meet treatment objectives. The height, location, and shape of the upper aligner blocks are customized to a patient's treatment goals. On the lower aligner tray, guide plates are placed to maximize contact area and lower material stress. The positioning of the blocks is determined during the treatment planning process.

4. Instructions for use

Indicated for use in alignment of teeth through orthodontic treatment of malocclusion.

5. Instruction for use

Your doctor's customized treatment plan is based on your doctor's prescription. Please read these directions carefully before use. Wear your aligners per your doctor's instructions, usually 22 hours a day or more.

a. Wearing procedure

1) Cleaning your aligner:

A cotton swab or a soft-bristled toothbrush is recommended to clean your aligner. Dip the toothbrush in water or special mouthwash to wipe the inner and outer surfaces of the aligner. Then rinse with water. The aligner should not be soaked in hot water or hot disinfectant water to avoid aligner deformation.

2) Clean your mouth:

Brush your teeth and rinse your mouth with water.

3) Clean your hands:

Wash your hands before handling your aligners.

4) Inserting your aligners:

INSTRUCTIONS FOR USE

Make sure you have the correct aligner. The "U" denotes upper teeth or "L" for lower teeth. This is followed by a stage number. Example: "U 1" means the first stage of upper, and "U 1-A" means the first aligner of the first stage of upper teeth. Fit your aligner on anterior teeth first, then posterior teeth.

Procedure:

- First, the anterior area of aligner is pressed onto the anterior teeth area of the aligner.
- Then the posterior area of aligner is pressed onto the posterior teeth area.
- After the appliances are fit in, various types of occlusion can be conducted to perfectly fit the appliances with teeth.

If part of appliance and appliance are not perfectly fit, the patient can use this part of teeth to bite medical cotton roll or gum, and help the appliance in good fit.

- A6: Ensure the block placement is adequate and appropriate for patient dentition, as planned during the treatment planning process. The blocks should be symmetrical to control the occlusal force direction in occlusal inclined planes. The blocks should exert gentle pressure on the jaw during biting and talking. The blocks should maintain the mandible in a more optimal bit position.

b. Removing procedure

Principle: Remove the appliance from posterior teeth to anterior teeth, from lingual to buccal region, and alternately on both sides.

Procedure:

- Maxillary appliance: The maxillary appliance can be removed from one side to the other. First, clamp the lingual side of one side's posterior appliance with fingers. Then pull the lingual edge of posterior appliance downward and buccally simultaneously. Both lingual and buccal side of appliance are pulled alternately until the appliance is detached from the teeth.
- Mandible appliance: The mandible appliance can be removed either from one side to the other or both sides simultaneously. First, clamp the buccal side of one side's posterior appliance with fingers. Then pull the buccal edge of posterior appliance upward and buccally simultaneously. Both lingual and buccal side of appliance are pulled alternately until the appliance is detached from the teeth.

c. Method of preservation

If necessary, the appliance can be removed in occasions including, eating, brushing teeth and social occasion. The appliance should be kept in a cleaned box after cleaning. It needs to be cleaned again before use.

d. Wearing time

- 1) Each stage of appliances should be worn for more than 22 hours per day for at least 10 days to achieve effective teeth movement. When multiple appliances made of the same geometry are used (Pro), the sum of their use time should at least 10 days per stage.
- 2) Each stage appliances should be worn at least 10 days. Before moving to a new aligner, your present aligners should fully fit over the teeth, be easily removed and no orthodontic force should be felt. It is recommended to change into new aligners at night prior to bedtime.
- 3) If the patient is unable to wear the appliances as prescriptive information, the patient should contact their doctor as soon as possible. To adjust the prescriptive wearing time, the doctor's consent must be obtained to continue the treatment. The treatment plan shall be followed, or the treatment shall be deemed as self-termination.

e. Dispensing the Product

- 1) When dispensing the product, the clinician must ensure the following:
the correct aligner is being dispensed the aligner properly fits and there are no gaps between the aligner and the patient's dentition.

f. Fit and Follow-Up

- 1) The patient shall be fitted by the clinician and follow regular return visits according to the doctor's request to investigate the function and progress of the treatment.
- 2) The current appliances should be worn at the time of return visit.

6. Safety considerations

- a. The appliances shall be used under the guidance of the attending doctor, and the scheme for its use shall be formulated by the attending doctor.
- b. The appliances are made for a particular patient, at a particular stage and in a correct sequence. The appliances must be used in order determined by the orthodontics sequence and can only be used for that particular patient.
- c. In the use of the appliances for orthodontic treatment, the patient may experience a certain level of discomfort. This is considered normal. If there are questions or prolonged discomfort, the patient should contact the treating doctor immediately.
- d. If there is discomfort when wearing the appliances, patient should not adjust the appliances, but contact the doctor as soon as possible.
- e. After receiving the appliances, the patient shall wear it in time to prevent unintended tooth movement.
- f. If the patient has one or more missing teeth, he or she must wear a retaining appliance in time.
- g. If the patient is unable to wear the clear aligners for the recommended time, does not comply with the doctor's instructions for wear, cannot complete return visits on time, there is the presence of abnormal shape of teeth, or there is new tooth growth in the course of treatment, the orthodontic treatment and timing may be affected.
- h. Some medical conditions and the use of drugs may affect the treatment.
- i. Malocclusion, severe overbite, and mixed dentition may require treatment other than bracket free clear appliances.
- j. The combination of other fixed/active orthodontic appliances may be required for complex treatment options that are not fully competent with the clear orthodontic appliances without brackets.
- k. Severe dentition congestion or occlusal imbalance may require the intervention of oral surgery. If this type of surgery is needed, the corresponding risks should be considered before treatment.
- l. Unconventional shapes, sprouting or missing teeth may affect the wearing of orthodontic appliances or may affect the treatment.
- m. The tooth with fillings or restorations may be displaced, and the fillings or restorations may need to be replaced
- n. The appliances do not move the dental implant effectively.
- o. In cases of traction accessories, the patients are required to cooperate with the traction according to the doctor's instructions.
- p. The functional accessories, may fall off because of numerous reasons, including improper intraoral habits, malocclusion, improper usage, etc. When it happens, please contact your doctor as soon as possible. On rare occasions, the falling part may be swallowed or breathed in. In case it happens, please seek treatment as soon as possible.
- q. Please avoid affecting the use of appliance because of improper Intraoral habits, malocclusion, improper usage, etc. When the appliance is fractured locally or damaged, please contact your doctor as soon as possible. When the surface of appliance is peeled off or abased, please also contact your doctor as soon as possible. For other cases or questions, please refer to your doctor.
- r. If the appliance cannot be engaged, please contact your doctor as soon as possible.
- s. In case of damage to the oral mucosa caused by wearing the appliances, the patient should contact the doctor as soon as possible and no self-adjustment shall be allowed.
- t. In the case of gum injury, oral ulcers and abnormal loosening of teeth caused by the non-appliances itself, patient should contact the doctor as soon as possible. If necessary, stop using the orthodontic appliances and wait for the tissue to recover before continuing to correct.
- u. When the appliances are damaged, deformed or lost, patient should contact the doctor as soon as possible and take prompt remedial measures.
- v. Angel Aligner with mandibular advancement features is neither tested nor approved for treatment of respiratory disorders, sleep apnea, and/or temporo-mandibular disorders (TMD).
- w. In rare instances, problems in the temporo-mandibular or jaw joint (temporo-mandibular disorder or dysfunction (TMD)) may result in joint pain, headaches, or ear problems. For patients in treatment with mandibular advancement features such as Occlusal Block (which are used to shift the lower jaw forward), problems in the jaw joint may be exacerbated.

INSTRUCTIONS FOR USE

7. Precautions

- Patient should not wear appliance to drink hot tea, drinks, etc.
- After eating or drinking sugary drinks, patient should take oral cleaning measures (such as gargling with water, brushing their teeth, etc.) before wearing the appliances.
- Patient should not use hot or disinfecting water to soak the appliances to avoid deformation.
- Patient should contact the doctor as soon as possible for the excessive irritation to the oral mucosa caused by wearing the appliances, and no self-adjustment by patient should be allowed.
- In the case of gum injury, oral ulcers and abnormal loosening of teeth caused by the non-appliances itself, patient should contact the doctor as soon as possible. If necessary, stop using the orthodontic appliances and wait for the tissue to recover before continuing to correct.
- When the appliance(s) are damaged, deformed or lost, patient should contact the doctor as soon as possible. Replacements should be ordered as soon as possible.

8. Contraindication

Clear aligners is contraindicated for patients with:

- Serious progressive periodontal disease.
- Known TPU/PETG/PE allergies.

Warning: Patients using Clear aligners should be able to understand and follow any treatment instructions as provided by their doctor.

9. Warnings

If an allergic reaction occurs as a result of using the product, have the patient discontinue use. Use of these aligners may cause gingival or dental soreness, as well as discomfort.

The A6 clear aligners (aligners with mandibular advancement blocks) may only be used in patients with growing mandibles presenting with retrognathic Class II malocclusions in permanent dentition or stable late mixed dentition, as determined by the clinician.

10. Storage Condition

The product should be stored in a non-corrosive gas, cool and clean environment.

11. Maintenance

After each use, use a cotton swab to dip water to clean the inside and outside surfaces of the appliances, and then put the appliances into a clean case for storage. When the appliances are damaged, deformed or lost, patient should contact the doctor as soon as possible and take prompt remedial measures.

12. Product Wear Cycle

It is recommended that each stage of appliance should be worn for at least 10 days. When multiple appliances made of the same geometry are used (pro), the sum of their use time should be at least 10 days for each stage.

13. Label Description

The "U" in the "Stage" denotes the maxilla (upper); "L" means the mandible (lower). Example: "U 1" means the first stage of maxillary.

14. Waste & Disposal

The appliance is eco-friendly and does not contain environmentally harmful substances. When handling used appliances, please refer to your doctor and follow your local waste & disposal policy.

15. Date of Production

See product packaging label for details.

16. Term of use

Under proper storage condition, the appliances have a shelf life of 3 years after produced.

17. Recommended Software

The software recommended for the creation of the invisible orthodontic aligners, as well as the treatment planning with the invisible orthodontic aligners is iOrtho software, by Shanghai EA Medical Instruments Co. Ltd.

18. Symbol Meaning

Symbol	Introductions	Symbol	Introductions
	Batch Code		Use-by date
	Caution		Consult instructions for use or consult electronic instructions for use
	Medical Device		Manufacture Date
	Manufacturer Name and Address	Rx only	Prescription Use Only
	Model Number		Non sterile
	Patient Number		Unique device identifier

Manufacturer: Wuxi EA Medical Instruments Technologies Limited.

Registered address: No.1619, Huishan Avenue, Huishan Economic Development Zone, Wuxi, Jiangsu Province, China.

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Fabricante: Wuxi EA Medical Instruments Technologies Limited.