PATIENT USE AND CARE INSTRUCTIONS

CONTENTS

- **English** Version ----- 01 - 10
- Version **Española** ----- 11 - 20

Patient Use and Care Instructions

CONTENTS

What is Angel Aligner?	02
Read before use	02
- Indications for use	02
- Contraindications	02
- Patient Instructions	02
- Safety Considerations	02
Wearing Angel Aligners	04
- Preparation before aligner insertion	04
- Aligner Insertion	04
- Aligner Removal	05
- Attention	06
Care and Maintenance	07
- Aligner Storage & Cleaning	07
- Aligner Wearing Time	07
- Office Visit Requirement	08
Q&A	08
Symbols Glossary	10

WHAT IS ANGEL ALIGNER?

The clear aligner system consists of a series of doctor-prescribed, thin, clear plastic removable orthodontic appliances (aligners) and proprietary 3D software. The aligners gently move the patient's teeth in small increments from their original state to a more optimal, treated state.

The traction accessory, also called "angelButton", made of thermoplastic polyurethane can be selected and will be bonded to the outer surface of the aligner by adhesive. A6 Angel Align is a design option for your aligners that adds blocks around the back teeth in your mouth to reposition your jaw during treatment. The blocks will guide your jaw into the optimal position while using the aligners.

The attachment template is a transparent aligner with a recess inside for filling the resin. The Doctor is able to accurately attach the attachment to the tooth, ensuring that the appliance applies force effectively and achieves the desired tooth movement.

READ BEFORE USE

Indications for use

Indicated for the alignment of teeth through orthodontic treatment of malocclusion.

Contraindications

Angel Aligners is contraindicated for patients with:

- Serious progressive periodontal disease
- Known TPU/PETG/PE allergies

Warning: Patients using Angel Aligners should be able to understand and follow any treatment instructions as provided by their doctor.

Patient Instructions

- Aligners are intended to be removed for eating, drinking, and oral care.
- After eating or drinking sugary drinks, patient should take oral cleaning measures (such as gargling with water, brushing their teeth, etc.) before wearing the appliances.
- Patient should not use hot or disinfecting water to soak the appliances to avoid deformation.
- Patient should contact the doctor as soon as possible for the excessive irritation to the oral mucosa caused by wearing the appliances, and no self-adjustment by patient should be allowed.
- In the case of gum injury, oral ulcers and abnormal loosening of teeth caused by the appliance itself, patient should contact the doctor as soon as possible. If necessary, stop using the orthodontic appliances and wait for the tissue to recover before continuing to correct.
- When the appliance(s) are damaged, deformed or lost, patient should contact the doctor as soon as possible.

Safety Considerations

- a. The appliances shall be used under the guidance of the attending doctor, and the scheme for its use shall be formulated by the attending doctor.
- b. The appliances are made for a particular patient, at a particular stage and in a correct sequence. The appliances must be used in


- order determined by the orthodontics sequence and can only be used for that particular patient.
- c. In the use of the appliances for orthodontic treatment, the patient may experience a certain level of discomfort. This is considered normal. If there are questions or prolonged discomfort, the patient should contact the treating doctor immediately.
 - d. If there is discomfort when wearing the appliances, patient should not adjust the appliances, but contact the doctor as soon as possible.
 - e. After receiving the appliances, the patient shall wear it in time to prevent unintended tooth movement.
 - f. If the patient has one or more missing teeth, he or she must wear a retaining appliances in time.
 - g. If the patient is unable to wear the clear aligners for the recommended time, does not comply with the doctor's instructions for wear, cannot complete return visits on time, there is the presence of abnormal shape of teeth, or there is new tooth growth in the course of treatment, the orthodontic treatment and timing may be affected.
 - h. Some medical conditions and the use of drugs may affect the treatment.
 - i. Malocclusion, severe overbite, and mixed dentition may require treatment other than bracket free clear appliances.
 - j. The combination of other fixed/active orthodontic appliances may be required for complex treatment options that are not fully competent with the clear orthodontic appliances without brackets.
 - k. Severe dentition congestion or occlusal imbalance may require the intervention of oral surgery. If this type of surgery is needed, the corresponding risks should be considered before treatment.
 - l. Unconventional shapes, sprouting or missing teeth may affect the wearing of orthodontic appliances or may affect the treatment.
 - m. The existing tooth patch may be displaced or may need to be replaced.
 - n. The appliances do not move the dental implant effectively.
 - o. In cases of traction accessories, the patients are required to cooperate with the traction according to the doctor's instructions.
 - p. The functional accessories, may fall off because of numerous reasons, including improper Intraoral habits, malocclusion, improper usage, etc. When it happens, please contact your doctor as soon as possible. On rare occasions, the falling part may be swallowed or breathed in. In case it happens, please seek treatment as soon as possible.
 - q. Please avoid affecting the use of appliance because of improper Intraoral habits, malocclusion, improper usage, etc. When the appliance is fractured locally or damaged, please contact your doctor as soon as possible. When the surface of appliance is peeled off or abased, please also contact your doctor as soon as possible. For other cases or questions, please refer to your doctor.
 - r. If the appliance cannot be engaged, please contact your doctor as soon as possible.
 - s. In case of damage to the oral mucosa caused by wearing the appliances, the patient should contact the doctor as soon as possible and no self-adjustment shall be allowed.
 - t. In the case of gum injury, oral ulcers and abnormal loosening of teeth caused by the non-appliances itself, patient should contact the doctor as soon as possible. If necessary, stop using the orthodontic appliances and wait for the tissue to recover before continuing to correct.
 - u. When the appliances are damaged, deformed or lost, patient should contact the doctor as soon as possible and take prompt remedial measures.
 - v. For aligners with the A6 mandibular advancement feature, biting directly on the blocks or positioning them on top of each other may increase the chance of aligner deformation.
 - w. Prior to each insertion and before bed, inspect aligners for deformation. Aligners with cracks should not be used. Note: Parent or guardian should inspect aligners of pediatric patients.

WEARING ANGEL ALIGNER

• Preparation Before Aligner Insertion


1. Check the label on the back of the packaging prior to inserting the new aligner. (Figure 1)
2. Wash your hands before touching the aligner.
3. Use a soft bristled toothbrush to gently clean the inside and outside surface of the aligner using cold water or mouthwash. Then rinse the aligner with water. Do not use hot water or disinfectant to clean the aligner as this may lead to deformation of the aligner.
4. Brush your teeth and rinse with water before inserting aligners – in particular after eating or drinking.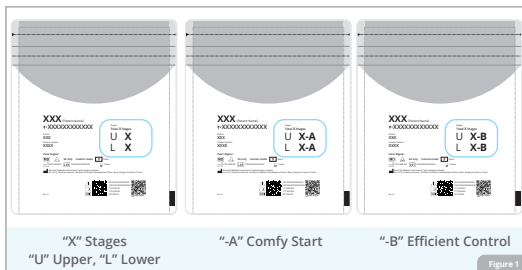

• Aligner Insertion:

1. You may insert either the upper or lower aligner first. Insert the aligner by positioning the aligner over the front teeth first. Once the aligner is in place, use gently pressure and guide the aligner over the teeth moving towards the back until the aligner clicks into place. (Figure 3)

If using A6, your doctor will help ensure the blocks are positioned correctly the first time. The blocks should exert gentle pressure on the jaw during biting and talking.

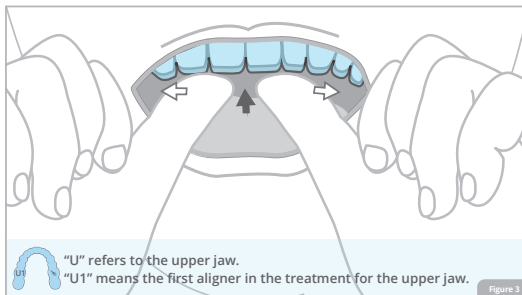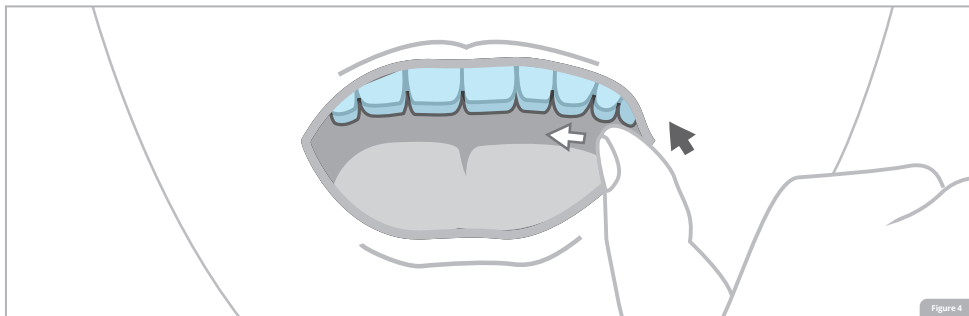
Figure 4

2. After pressing, check to make sure the bite between the upper and lower jaws is fine and that the aligner fits on all the teeth. (Figure 4)
3. If the aligners do not seat completely, use the chewies provided in your Starter Kit to help guide them into place. If aligners do not fit at all, consult your doctor for further instruction.

• Upper Aligner Removal:

Starting at the back of the mouth, use your finger to pull the inside edge of the aligner gently away from the teeth. Then alternate pulling between left and right side of the upper aligner until it is removed. (Figure 5)


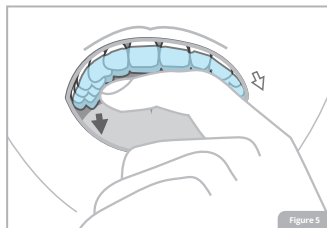
Figure 5

• Lower Aligner Removal:

Starting at the back of the mouth, use your finger to push the outside edge of the aligner gently away from your teeth. Then alternateate the pushing between the left and right sides of the aligner until it is removed. (Figure 6)

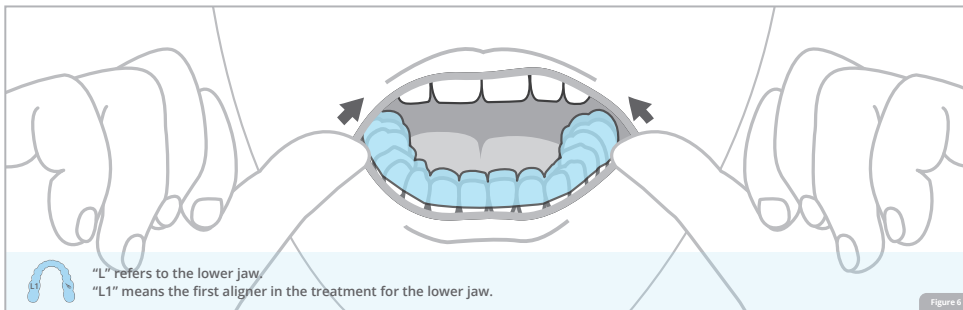• Attention:

1. At the beginning of wearing new aligners, you may feel slight discomfort or soreness. This is a normal reaction of your teeth to orthodontic treatment. If you feel sharp pain or discomfort, please contact your doctor.
2. If you have difficulty in inserting or removing your aligner, you can use your nail to gently lift the edges of the aligner in the area where it's hard to engage or remove. Then gently press the aligner to fit or wriggle slowly to remove. Do not use too much force, as this can cause the aligner material to fracture or distort.
3. Prior to inserting your aligners in, inspect them for cracks or deformities. If your aligner is damaged, distorted, or lost, please contact your doctor as soon as possible for a replacement and for further direction to avoid excessive treatment delay.
4. If your gingiva (gum tissue) becomes irritated from the aligners, do not try to modify the aligners. Contact the doctor as soon as possible for further instruction. If during treatment, you experience gingiva damage, dental abscess or loosening of teeth, contact the doctor as soon as possible for further instruction. If necessary, put treatment on hold and resume when fully recovered.

CARE AND MAINTENANCE

• Aligner Storage and Cleaning

1. Aligner storage case(s) are included in your Starter Kit. We recommend that you store your Angel Aligners in a case when they are not in your mouth. This will help protect them from loss and damage. Keep aligner cases clean by hand washing them with soap and water.
2. If aligners need to be removed to eat or drink, please rinse your aligners and place them in the provided case for safe keeping. (Figure 7)
3. Use a soft-bristled tooth brush and gently brush the inner and outer surface of the aligner with water. Do not use hot water or disinfectant to clean the aligner as this can cause deformation or discoloration.

• Aligner Wearing Time:

1. Wear your aligners for 22 hours or more per day or as directed by your doctor.
2. Each set of aligners is designed to be exchanged with the next step aligner after at least every 10 days or as instructed by your doctor. It is recommended to switch to new aligners before sleeping.
3. If for any reason you are not able to wear your aligners for the recommended 22 hours (or as directed by your doctor) you may need to extend the number of days, the aligner step is worn. Consult your doctor for further instruction.

- **Office Visit Requirements:**

Office visits are often necessary for effective treatment and progress tracking. If you miss an appointment, call to reschedule as soon as possible.

Please wear your aligners when you visit your doctor and bring your last 2 steps of aligners with you to your appointments unless otherwise directed by your doctor.

Q&A

Q: Why am I experiencing soreness?

A: Many patients will feel soreness for the first couple of days after starting each new aligner step. This is completely normal. This indicates that the aligner is applying force to your teeth and moving your teeth towards the desired position. This force is very light and will not cause any damage to your teeth. The soreness will generally reduce after wearing for a few days.

Q: Are there restrictions on what I can eat?

A: In general, no. Unlike traditional orthodontics, you can usually eat and drink whatever you desire because you remove your aligners while eating. Thus, there is no need to restrict your consumption of any of your favorite foods and snacks, unless otherwise instructed by your doctor.

Q: When wearing aligners, can I drink hot or cold drinks?

A: Drinking anything besides cool water while wearing aligners is not recommended. Keep aligners away from alcohol and hot, sweet, or colored liquids which can damage the aligner and cause cavities or white spot lesions on teeth.

Q: Will Angel Aligners affect my speech?

A: Like any other orthodontic treatment, our customized aligners may affect your speech for the first 1-2 days. Your speech will typically return to normal once you become adjusted to it.

Q: As the aligner is made of plastic, will it be damaged easily?

A: Angel Aligners are made of clear, durable, and medical grade plastic. While the aligners are strong and flexible, ensure that you are using them correctly, in accordance with the use instructions and your doctor's directions.

Q: What do I do if I lose or damage my aligner?

A: If you lose or damage your aligner, please contact your doctor as soon as possible. Your doctor is likely to tell you to move back to your last pair of aligners – this is why it is very important to keep your last 2 sets of aligners.

Q: For patients undergoing mandibular advancement, why do some of my aligners have 'blocks' on them?

A: The 'blocks' on the sides of aligners are called precision wings, and are especially designed to position the lower jaw forward to help correct the bite.

Q: Why do some of my aligners have bumps or ridges on them?

A: Depending on your specific treatment, some of the movements may require either "attachments" or "ridges" to help the aligner grip the teeth. These bumps, or wells, are where the aligner grips the attachment the doctor places on your teeth. The attachments are actually small pieces of composite the doctor affixes to your teeth that are then gripped by bumps on your aligners. The ridges are slim indentations in your aligners. Your doctor will use one or both of these features to attempt to achieve the desired movement.

Q: What if I lose or break an attachment?

A: In the event that an attachment is lost or broken, you should immediately contact your doctor.

Note: If you have additional questions after reading this pamphlet, please consult with your doctor.

Symbols Glossary

Symbol	Introductions	Symbol	Introductions
	Batch Code		Use-by Date
	Caution		Consult instructions for use or consult electronic instructions for use
	Medical Device		Manufacture Date
	Manufacturer Name and Address	Rx only	Prescription Use Only
	Model Number		Non Sterile
	Patient Number		Unique Device Identifier

Manufacturer: Wuxi EA Medical Instruments Technologies Limited.

Registered address: No.1619, Huishan Avenue, Huishan Economic Development Zone, Wuxi, Jiangsu Province, China.

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Fabricante: Wuxi EA Medical Instruments Technologies Limited.