Retainer

INSTRUCTIONS FOR USE

CONTENTS

- **English** Version ----- 01 - 04
- Version **Française** ----- 05 - 08

INSTRUCTIONS FOR USE

English Version

1. Model and specification

This manual applies to the product model Retainer. Because the product is custom-made after molding, the size varies.

2. The main structure and composition of the product

This product is made of clear thermoplastic polyurethane material, and the internal contour of the product is consistent with the crown morphology of all or part of the patient's teeth.

3. Product performance

In the state of use, the product covers the crown of the user's teeth. By means of the corresponding tooth shape on the product and the actual position of the corresponding teeth on the user's actual dentition, the teeth can be prevented from moving, so as to achieve the treatment results after the completion of orthodontic treatment.

4. Instructions for use

The clinician should guide the patient to take off and wear the product correctly after being familiar with the usage and ask the patient to revisit for confirmation if necessary.

a. Wearing notice

1) Cleaning:

Use a soft toothbrush in clean water, wipe the inside and outside surfaces of the product, and then rinse with clean water. Do not soak in hot water or disinfectant water to prevent deformation of the product.

2) Oral cavity cleaning:

brush teeth and rinse mouth with water before wearing the product.

3) Please clean your hands before taking off and putting in the product.

4) Wearing method:

Principle: First wear the anterior teeth area (anterior region), then the posterior teeth area (posterior region).

Method:

- First insert the anterior region of the product to make it fit with the teeth, then press and gradually insert the posterior region.
- After pressing well, biting the upper and lower dentitions tightly the product will be completely in place and fully fit with the teeth surface.

If there is a slight unfitting between the teeth and the product in the local area, bite the medical cotton roll or chewy in the area to

INSTRUCTIONS FOR USE

assist in positioning.

5) Taking off method:

Principle: Posterior to anterior, inside to outside, alternating sides.

Method: To take off the upper retainer, grasp its inner (palatal) edge of the posterior teeth area with your fingers, and gently pull downward and outward, alternate on both sides until it is detached from the teeth. To take off the lower retainer, press your thumb nail against its outer (buccal) edge of the posterior teeth area and push upward, alternate on both sides until it is detached from the teeth.

6) Storage method: The product should be removed during eating, brushing teeth and can be removed for short social occasions. The product should be kept in a cleaned case after cleaning. It needs to be cleaned again before use.

7) If the oral mucosa is hurt after wearing this product, contact your attending doctor as soon as possible and do not adjust it by yourself.

b. Wearing time

1) Wear this product for more than 22 hours a day during normal use.

2) If the user cannot wear according to the wearing time requirements, he should inform the attending doctor in time and formulate a treatment plan and follow the doctor's medical advice.

c. Follow-up request

1) The user should make regular follow-up visits as required by the attending doctor to confirm the maintenance effect.

2) Wear this product during follow-up visits.

5. Safety Considerations

a. This product must be used under the guidance of the attending doctor.

b. This product is custom-made for a specific patient and can only be used for his/her usage.

c. When wearing this product, if you feel uncomfortable, do not adjust by yourself, contact the doctor as soon as possible.

d. After receiving this product, the patient should wear it promptly to prevent unexpected tooth movement.

e. If the user does not wear the product as required, or does not follow the doctor's advice, or fails to return to the attending doctor on time, or teeth erupt during treatment, it may affect the treatment maintenance after the completion of orthodontic treatment.

f. Certain medical conditions and drug use may affect the efficacy of this product.

g. Unconventionally shaped, erupting, or missing teeth may affect the wearing of this product and may affect the intended efficacy.

h. In rare cases, temporomandibular joint related issues may cause joint pain, headaches, or ear diseases. Consult your attending doctor if these conditions occur during use.

i. Avoid bad oral habits, abnormal bite, improper use and other factors affecting the use of this product. If there is a local collapse or fracture of the product, contact the attending doctor promptly. If there is local peeling or wear on the surface of the product that does not affect the function of the product, it can be used normally. If you have any other questions or concerns, please consult the

INSTRUCTIONS FOR USE

attending doctor.

- j. If this product is not in place during use, consult the attending doctor.
- k. If there is a gingival damage, oral ulcers, abnormal tooth mobility, etc., you should also contact the attending doctor as soon as possible. Suspend the use of this product when necessary, and continue to use after the tissue recovery.
- l. If the product is damaged and can no longer be used, or is lost, contact the attending doctor as soon as possible and take remedial measures in time.

6. Precautions

Do not wear this product while drinking hot tea, beverages, etc.

After eating or drinking sugary beverages, users should take oral cleaning measures (such as rinsing with water, brushing teeth, etc.) before wearing this product.

7. Indications for Use

It is used to maintain the treatment results after the orthodontic treatment.

8. Contraindication

The product is contraindicated for patients with:

- Serious progressive periodontal disease.
- Known TPU/PETG/PE allergies.

Warning: Patients using the product should be able to understand and follow any treatment instructions as provided by their doctor.

9. Care and maintenance

After each use, clean the inside and outside surfaces of the product with a toothbrush, and then store the product in a clean case.

10. Storage condition

The product should be stored in a cool and clean environment without corrosive gas.

11. Production date

See the packaging label inside the product for details.

12. Product wearing cycle

Wear this product for the duration prescribed by the doctor. It is recommended that the single set be worn no more than 6 months.

13. Service life

3 years.

14. Label description

The "U" in the label represents the upper retainer; "L" means the lower retainer.

15. Disposal and treatment

This product has no ecological toxicity and does not belong to harmful environmental substances. For waste products after use, please consult the attending doctor or follow the relevant local regulations for waste disposal.

16. Symbol Meaning

Symbol	Introductions	Symbol	Introductions
	Batch code		Use-by date
	Caution		Consult instructions for use or consult electronic instructions for use
	Medical device		Date of manufacture
	Manufacturer	Rx only	Prescription Use Only
	Model number		Non-sterile
	Patient number		Unique device identifier

Manufacturer: Wuxi EA Bio-Tech Co., Ltd.

Address: No.36 Guanshan Road, Xinwu District, Wuxi, Jiangsu Province, China

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Fabricant: Wuxi EA Bio-Tech Co., Ltd.